

Aster Propphy



PRODUCT DESCRIPTION:

ASTER PROPHY contains abrasives to maximize the removal of plaque without appreciable abrasion of enamel. The thixotropic properties are controlled in a manner to ensure easy transfer to cup or brush without undesirable spattering at the rotation speeds used in prophylaxis. ASTER PROPHY contains no fluoride.

INDICATIONS:

Use for the removal of plaque from patient's teeth & polishing of tooth surfaces.

DIRECTION FOR USE:

ASTER PROPHY is thixotropic in nature and more fluidity can be attained when subjected to rotation. Tighten cap properly after dispensing. For optimum results use a prophy type handpiece and rubber cup/bristle brush combination. Fill rubber cup/ bristle brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup/ bristle brush goes below the free margin of the gingiva into the gingival sulcus. Repeat this procedure for each tooth until all exposed surfaces have been polished. Unwaxed dental floss may be used to remove any excess paste through the interproximal areas.

INDICATIONS:

Tighten cap of the tube properly after dispensing.

STORAGE:

Store in cool and dry place between 50C to 250C

PACKAGING:

Pack of 45 g tube Medium grit- Lemon Yellow